



Value Life

Operating Room  
Patient Advice Sheet



**INDERMIL® flexifuze™**  
The next generation  
tissue adhesive

These sheets have been printed using environmentally-friendly processes, on paper from well-managed sources, and can be recycled. If you would like to receive an electronic copy, please contact the Vygon Marketing Team on [marketing@vygon.co.uk](mailto:marketing@vygon.co.uk).

This pad contains 50 individual sheets

Designed and Printed by Vygon (UK) Ltd  
Vygon (UK) Ltd, The Pierre Simonet Building, V Park, Gateway North, Latham Road, Swindon, Wiltshire SN25 4DL  
Tel: 01793 748800 Fax: 01793 748899 Web: [www.vygon.co.uk](http://www.vygon.co.uk) Email: [vygon@vygon.co.uk](mailto:vygon@vygon.co.uk) [@vygonuk](https://twitter.com/vygonuk)  
Copyright Vygon (UK) Ltd 2016 Content correct as of 02/2016 - Code: DXJB0006001

# Patient Advice

Your wound has been closed with INDERMIL® flexifuze™ tissue adhesive. The following guide will help you to understand this method of wound closure.

## 1. Product Information

- INDERMIL flexifuze tissue adhesive is a special medical adhesive which has been designed to close wounds. It can be used instead of conventional stitches and skin staples.
- The surface of the wound which has been closed using INDERMIL flexifuze tissue adhesive will look different to a wound which has been closed by conventional stitches.
- The adhesive has been applied to the surface of the skin and may take on an irregular crinkled appearance, this is quite normal and will not affect the function of the adhesive.
- The overall appearance can best be described as resembling a dry scab and should be treated like a scab – **do not pick!**



## 2. Care and Maintenance

- INDERMIL flexifuze is shower-proof, enabling patients to shower as soon as the adhesive has dried\*.
- Avoid scrubbing the wound site.
- Do not apply any medications or cream to the wound. Do not pull or pick at the wound or bandage.
- Report any discomfort or other concerns regarding your wound to your doctor.



## 3. Final Results

- The adhesive will come away from the skin of its own accord within about five to eight days, just like a normal scab. By this time the wound should be sufficiently healed.
- There is no need to seek further medical assistance unless you are unhappy with your wound.
- The scar will take approximately six months to fade.



\* Though the adhesive is shower-proof, excessive contact with water may affect its integrity.